

2Care TEAM OPERATOR INSTRUCTIONS



What is it

• Management software to address needs of the team dedicated to Vascular Access according to the role:

osinters

- Responsible
- Team staff
- Department Operator



- REPORTING Vascular Access
- COMMUNICATION and interaction between team and operational unit with schedulable consultancy
- INSERTION of complications and their prevention
- AGENDA to plan events
- DRESSING REGISTRATION
- REGISTRATION consultancy
- LESS administrative work, MORE care time
- AVAILABILITY of data in real time



MAIN DASHBOARD

- Personal login
- Registration
- Confirm with email
- Dashboard
- Featured activities
- Personalized



Vascular Access request list

- Buttons for quick filter
- Sorting and searching on all columns
- Export to Excel
- Insert new request

≡ Ω2care	Type som	e text to se	earch for	9							MENU	+ 🌒 HEL	LO MARINE COMPANY	+ E
Calendar	LIST O	F VAS	CULAR A	CCESS REQU	JESTS								+ NEW REQU	EST
Vascular access list					TUTTI (16)	TO BE CONF	RMED (10)	CONFIRMED (1)	SCHEDULED (5)	CANCELE	D (62) □ → ▼	Q Search		^
Vascular access request list		ld ≖	Status 👳	Scheduled =	Request = Date	Expected = date of use	Catheter =		Operator 👳 Su	rname = d name	Date of Birth	Patologia 👳	Request = manager	Ap tel
Operators		Q	Q	Q -	Q -	Q -	Q	Q	Q Q		Q -	Q	Q	Q
Data extraction									· · ·					
Dressing material kit		5287	Confirmed		10/5/2022, 4:26 PM	10/30/2022	PICC		FR	ANCO	11/22/1942	K COLON	DOTT. SSA	34
New request for vascular access														
New removal		5316	Scheduled	11/10/2022, 5:30 PM	10/21/2022, 2:56 PM	11/7/2022	PICC		LIL	IANA	12/12/1943	K OVAIO	luigi seri	07
Late Complications			To be		10/27/2022 0:46							K		
Departments		5326	confirmed		AM	11/15/2022	PICC		SA	RA	3/18/1982	MAMMELLA		39:
	D	5329	To be confirmed		10/27/2022, 1:51 PM	10/30/2022	PICC		GI/	ANCARLO	12/10/1953	K POLMONI	DOTT. PAOLO	34. 07
		5331	Scheduled	11/10/2022, 6:00 PM	10/28/2022, 6:40 PM	11/10/2022	PICC	2	ZY	LFIE	12/18/1956	K COLON	Lucia Amico	34
		5333	Scheduled	11/9/2022, 2:30 PM	11/1/2022, 5:22 PM	11/2/2022	PICC		BR FR	UNO ANCESCO	1/24/1953	K PROSTATA	Dott. De Paolo	
														Ŧ
osinters 360° Copyright © 20	022 - osinters	s.r.l.									Useful lir	nks - Request assi	stance - Versione	2.0.0.0

PATIENT SEARCH



It is carried out for:

• New Plant / Plant from other health care companies

6) Check

- New Consultancy
- Company registry
- Database connection

Required fields (*)

	■ 2Care x + ← → C A A https://2care	coredemo.2care.cloud/Common/	Pazienti/EditPaziente?Destinazione+Paziente						Q 🔓	= G ¢ @	0 × ∰
	= 🖸 2care	Digita un testo da ce	ercare					MENU	1 💊 🜑	CIAO LUIGI.CENS	sori 🚦
	Nuova richiesta accesso vascolare	MODIFICA P	AZIENTE		<mark>7</mark>)(CONFIR	MATION D	ATA		i salva 🖿 A	INNULLA
	Operatori	DATI ANAGRAFICI							$\overline{}$		
	Pazienti							Ē			
	Calendario	Cognome *	ROSSI		\otimes	Sesso *	Maschio				
	Complicanze Tardive	Nome*	MARIO		\otimes	Codice fiscale	RSSMRA67B02F764S				8
	Parametri procedura	Data nascita *	•			Comune di nascita					
	Dipartimenti		• 2/2/1967 •	• ® •			MOSCIANO SANT'ANGEL	0		× ×	~
	Lista accessi vascolari	Note									
	Estrazione dati										
	Lista richieste accessi vascolari										_
		DATI RESIDENZA									~
data in s	ections	DAN DOMICILIO									~
											~
		ALTRI DATI PAZIENT	E								~
	osinters 360° Copyright © 20	21 - osinters s.r.l.							Richi	edi assistenza - Ve	rsione 1.0.0

•••	-	\equiv 🖸 2care	Type some text t	to search for	3				Ν	IENU 🕂 🍈 HELLO LUIGI	.CENSORI
Now nla	nt	Calendar	PLANT REG	QUEST						SAVE	r CANCEL
		Vascular access list Vascular access request list		Mobile 3387909427	942) Request 10/5/2022 Date 4:26:50 PM Plant date						ВМІ
	Contact informatio	N Operators		Email	Vascular access Confirmed						20.76 Normale
request		Data extraction		PICC	status	Pathology: K COLON					
request		Dressing material kit	PLANTING DATA								Â,
		New request for vascular access	Expected date of use *	S • 10/30/2022	• 🛞 -		Catheter type *	PICC		× ×	
		New removal					Catheter materials	Select		•	
		Late Complications	Informed patient	Si	× ×		Lights	1 lumi	· ·	_	
PLANT DATA		Departments	Information material delivered	Si	× ×		High pressures		Valved	Tunnelizable	ı 📕
			The patient cooperates	Si	× ×		Scheduled plant date	© ·)		
			Patient weight	60	× 🗘		Scheduled operator	Select		*	
Required fields (*)			Patient height	170	® 📜				<u> </u>		」 ノ
			APPLICANT DAT	A				A			
	Information group	ad by costions	INDICATIONS VA	SCULAR ACCESS							~
	information group	ed by sections	LABORATORY TE	STS							~
			PATHOLOGY								~ .
		OSINTERS 360° Copyright © 20	22 - osinters s.r.l.							ul links - Request assistance -	Versione 2.0.0.0
									ogra		
									t bu		
									lan		

New plant contact int request

• APPLICANT DATA

Required fields (*)

	= 💟 2care	Type some text to	o search for			MENU 🕂 🇌 HELLO	:
	Calendar	PLANT REC	UEST			SAVE 🗠 CANO	EL
	Vascular access list		Request 10/5/2022 Date 1/26/50 PM				
orn	Vascular access request list		Phone Plant date Vascular			B 20 Nori	иі 76 male
	Operators		PICC Pathology: K COLON				
	Dressing material kit	PLANTING DATA				~	·
	New request for vascular access	APPLICANT DATA					
	New removal	Unit *	DAY HOSPITAL ONCOLOGIA (Ospedale di Città di Castello (DEA)) 🛛 🗴 🗸	Request manager	DOTT. SSA AMICO	8	
	Late Complications	Patient provenance *	Structure O Domicile O Other structure	Room/Bed			
	Departments	Other structure of origin	Select	Applicant Email	locu.armodputantma1.0		
		Expected duration of treatment	6 Mesi - 1 Anno 🛞 🔻	Applicant telephone *	3481222033	8	
		Discharge with catheter	No 🛞 🕶	Transportable patient	8		
		Applicant Note					
		INDICATIONS VAS	SCULAR ACCESS			~	
		LABORATORY TES	STS			~~~~~~~~~~~~~~~~~~~~~~~~~~~~~~~~~~~~~~	
	osinters 360° Copyright © 20)22 - osinters s.r.l.				Useful links - Request assistance - Versione a	2.0.0.0

osinters 3



Ac

New plant request

• LABORATORY EXAMS

Required fields (*)

= 🖸 2care	Type some text to search for Q	:										
Calendar	PLANT REQUEST	CEL										
Vascular access list	FRANCO (11/22/1942) Mobile Request 10/5/2022											
Vascular access request list	Phone Plant date 200	MI 0.76										
Operators	Email	male										
Data extraction	PICC Pathology: K COLON											
Dressing material kit	PLANTING DATA											
New request for vascular access	APPLICANT DATA	_										
New removal	INDICATIONS VASCULAR ACCESS	-										
Late Complications	LABORATORY TESTS											
Departments	INFECTIONS											
	Infection* No Redditional details about infections HEMATOCHEMICAL TESTS											
	+ NEW BLOOD TESTS 🕒 🗸											
	\vec{1}{\vec{1}} Id Operator \vec{1}{\vec{2}} Date \vec{1}{\vec{2}} GB \vec{1}{\vec{2}} HB \vec{1}{\vec{2}} PT \vec{1}{\vec{2}} ATIII \vec{1}{\vec{1}} INR \vec{1}{\vec{1}} Fibrinogen \vec{1}{\vec{1}}											
	No data											
	PATHOLOGY											
osinters 360° Copyright © 20	2022 - osinters s.r.l. Useful links - Request and the - Versione	2.0.0.0										
	ds new											

New plant request

• PATHOLOGY DATA

Required fields (*)



osinters

New plant request

CLINICAL INFORMATION

Required fields (*)





Additio

New plant request

• VASCULAR ACCESS INDICATIONS

Multi

Required fields (*)

= 🖸 2care	Туре з	some text to searc	h for	Q		MENU	🕂 🌒 HELLO 🔙		
Calendar	PLAN	NT REQUES	ST		INTER		SAVE	r CANCEL	
Vascular access list		Mobi	le	ANCO (11/22/1942) Request 10/5/2022					
Vascular access request list		Phor	ne	Plant date				ВМІ 20.76	
Operators		Ema	ail ann ann	access Confirmed status				Normale	
Data extraction		PICC		Pathology: K COLON				-	
Dressing material kit	PLANT	FING DATA						~	
New request for vascular access	APPLI	CANT DATA						~	
New removal	PIDICA	ATIONS VASCULA	R ACCESS						
Late Complications	-	Id		Description	-	Note	-		
Departments		1		Idratazione quotidiana				1	
		2		Trattamenti Vescicanti - Chemioterapia Antineoplastica				1	
		3		NPT				1	
				Patrimonio Venoso Compromesso				1	
ole selection				Farmacoterapia				1	
		10		Perioperatorio				1	
		11		Infusione Emoderivati		4		1	
		13		Prelievi ega seriali				1	
		6		Dialisi trisettimanale			S	1	
osinters 360° Copyright © 20	22 - osint	ters s.r.l.				Useful lin		- Versione 2 0 0 0	
consulting JOU COPPLIGHT @ 20						osciul III		Control 2.0.0.0	



ENTER (at t

Plant request

ATTACHMENTS INSERTION

- Optional
- After confirming the request

+ NEW ATTACHMENT

	ADD ATTACHME	NT		
	SELECT A FILE	or Drop a file here	Ĉ	X CLEAR
	Туре	Indagine Strumentale		× •
	Description	Dematerialized recipe		
	Change file name			G
t	(optional)			
			:	SAVE CANCEL
VISITS			2	le end)

A VASCULAR ACCESS **REQUEST HAS** BEEN CREATED

Now in the next steps you can see the management, or how to:

- Programming Plants
- Enter Implant Report
- Inserting Dressings
- Inserting Consulting
- Inserting Late Complications
- Schedule Control medical examination
- Program Explants
- Record Device removals and Explants









Vascular Access Contact infor Reporting

• VESSEL

ar				nal d		o req										
	= 💆 2care	Type some text t	o search for	ata	0	luest						MENU	+ @	HELLO	CONSIDER	:
	Calendar	VASCULAR	ACCESS	REPOR	TING									SAVE	r CANCE	L
Contact informa	Vascular access list		Mobile	ROMANO (5/	(1/1941) Request Date 9, 5	/1/2022 :53:53 PM									BM	
ing	Operators		Phone 📞		Plant date 9, 6 /ascular access status A	/1/2022 :27:34 PM .ctive									20.9 Norm	4 ale
	Data extraction		PICC 4F (79 gior	ni)			Pathology: K POLMONI									_
	Dressing material kit	STATUS													~	_
	New request for vascular access	CATHETER													~	
	New removal	BLOOD VESSEL													^	١
	Late Complications	Pots *	Pagilion			~ -		Blood	vessel		- ^					
	Departments					· ·		diamete	er (mm)	6	× •					
		Blood Vessel type *	Vena			× •		Plar	it side -	SINISTRO	× •					J
		TECHNIQUE													~	
nformation grou	uped by sections	LABORATORY TE	STS												~	
inormation gro	apea by sections	EARLY COMPLIC	ATIONS												~	
		FURTHER													~	
	3200ml - 4															Ţ
	consulting 360° Copyright © 20	22 - osinters s.r.l.										Useful link	s - Reques	t assistance -	Versione 2.).0.0

Perso

Link t

osinters Consulting 360

Vascular			Personal	Link to re							
Access	= 🖸 2care	Type some text to	o search for	Q est						MENU 🕂 🌒 HELLO II	
Reporting	Calendar Cal	VASCULAR	ACCESS ROMANO Mobile t Phone t Email	RTING (5/1/1941) Request Date 9/1/2022 5:53:53 PM Plant date 9/1/2022 6:27:34 PM Vascular access status Active						SAVE	EMI 20.94 Normale
 TECHNIQUE Positioning Anchor Check 	Dressing material kit New request for vascular access New removal Late Complications Departments	STATUS CATHETER BLOOD VESSEL TECHNIQUE Attempts Positioning Time (min.) Positioning	1 15 Select	 ⊗ ⁺_↓ ⊗ ⁺_↓ 	Pathology: K POLMONI	Tur	Tunneled]	A *		~ ~ ~
	Multiselection	ANCHORING TECHN	IQUES Ţ	Description Punti di sutura SecurAcath® Suturless device	÷		Id $=$ 1 4	Description ECG RX TIP NAVIGATOR	Note $\overline{\nabla}$	ConteggioAllegati 1 0 0 0	/ ± / ± / ±
	osinters 360° Copyright © 20	22 - osinters s.r.l.								Useful links - Request assistant	e - Versione 2.0.0.0

osinters 360°



Vascular Access Reporting

• EARLY COMPLICATIONS



Vascular Access

• OTHER DATA

		Pe						Consulting 360°
		rson	nk to					
		al da	o rec					
🔲 💟 2care	Type some text t	o search					MEN	U 🕂 🌒 HELLO MARIN CARNICAN 🚦
lendar	VASCULAR	ACCESS REPOR					ENTER	SAVE 🗠 CANCEL
scular access list		ROSSI MĂRIO (5/1/1941) 33123456789	9/1/2022					V
ormation _{est list}		Mobile C	S:53:53 PM					BMI
erators		Phone	Plant date 9/1/2022 6:27:34 PM					20.94 Normale
ta extraction		Email	Vascular access status Active					
		PICC 4F (79 giorni)		Pathology: K POLMONI				
essing material kit	STATUS							~
w request for vascular access	CATHETER							~
w removal	BLOOD VESSEL							~
te Complications	TECHNIQUE							~
partments	LABORATORY TE	STS						~
	EARLY COMPLIC	ATIONS						~
	FURTHER							Â
	Plant location							
	Patient weight	Ambulatorio dedicato			Patient height		- 4	
		57	× _			165	× _	
	Reporting							
								J
intera								•
consulting 360° Copyright © 20	122 - osinters s.r.l.						Useful I	inks - Request assistance - Versione 2.0.0.0

nointono



ORIGIN

۵.

DRESSINGS



osinters Consulting

VIEW PREVIOUS DRESSING

= 🖸 2care	Type some text to	search for			MENU 🕂 🌒 HELLO MARINA :
Calendar	MEDICATIO	N INSERTION			🖬 SAVE 🖛 CANCEL
Vascular access list		ROSSI MARIO (5/1/1941) 33123456789 Mabile			
Vascular access request list		Mobile Request Date 5:53:53 PM 39123456789 010000			ВМІ
Operators		Phone Plant date 9/1/2022 6:27:34 PM			20.94 Normale
Data extraction		Email status Active PICC 4F (79 giorni) Pathology: K POLMONI			
Dressing material kit	PROVENIENZA				~
New request for vascular access		2200100			
New removal	VIEW PREVIOUS D	RESSINGS			Â
Late Complications	Date ExitSite	9/1/2022 6:35 PM	Operator	Lucia	
Departments		Cute sana ed integra: non segni di flogosi. Possibile iperpigmentazione cicatriziale			1
		ninitata an ninitettato perimetro denexit site.			
		Other ExitSites		Materials	
		No data		SECURACATH	
				TRASPARENTE IN POLIURETANO	
				CLOREXIDINA 2%	
				NEEDLE-LESS SYSTEM	
				PORT PROTECTOR	
	Note			dre	
		-		for	
osinters 360° Copyright ©	2022 - osinters s.r.l.			sed	Useful links - Request assistance - Versione 2.0.0.0
				u s	
				eria	
				late	
				2	

ENTER CURRENT EXIT SITE STATUS

\equiv 💟 2care Q MENU + M HELLO Type some text to search for ... **MEDICATION INSERTION** CANCEL Calendar SAVE ROSSI MARIO (5/1/1941) Vascular access list 33123456789 Request Date 9/1/2022 Mobile 5:53:53 PM Vascular access request list BMI 39123456789 Plant date 9/1/2022 20.94 Phone 6:27:34 PM Normale Operators Vascular access Email status Active **Data extraction** PICC 4F (79 giorni) Pathology: K POLMONI **Dressing material kit** URRENT EXITSITE New request for vascular access --Other ExitSites Image ExitSite New removal Cute sana integra Late Complications GRADO 0 Cute sana ed integra: non segni di flogosi. Possibile iperpigmentazione cicatriziale Decubito - Securacath Departments limitata all'immediato perimetro dell'exit site. Edema del punto di inserzione Area calda al tatto Iperemia < 1cm intorno all'exit-site eventualmente associata a presenza di fibrina. Single selection Grado di compromissione reversibile grazie all'utilizzo di medicazioni a lento rilascio di Emissione di secrezioni clorexidina 2%. Flebite GRADO 2 Iperemia > 1cm intorno all'exit site eventualmente associata a presenza di fibrina. \checkmark Fritema Indicata medicazione a lento rilascio di clorexidina 2%; tampone exit site se ritenuto opportuno. Prurito GRADO 3 Eruzione locale Iperemia associata a seccrezione, pus, eventualmente associata a presenza di fibrina. Necessari: tampone exit site, esecuzione emocolture da catetere e da vena periferica, MARSI attivazione consulenza medica per prescrizione terapia antibiotica empirica e valutazione ulteriori necessità. Lesioni Cutanee Tasca Port osinters 360° Copyright © 2022 - osinters s.r.l. Useful links - Request assistance - Versione 2.0.0.0 Multiselection

USED MATERIALS Multi-selection of materials used for dressing Material kit selection for quick insertion





IMAGE ANALYSIS

It is possible to insert photos from tablets and smartphones (apps) to visually document the status of the exit site in order to monitor and compare the effectiveness of treatments in the event of complications



DSinters Consulting 360°

A DRESSING HAS BEEN RECORDED

In the next steps it will be shown how:

- Enter Consulting
- Enter Late Complications
- Schedule catheter removal
- Record Removals and Explants

osinters 360°

FAST CREATION

Consulting registration





Late Compl record

COMPLICATION

nplications ording	Date Presumed Complication	Check box in case of Suspected complication		
	Noto			
2			SAVE	CANCEL
COMPLICATION CONSULTS REMOVAL ATTACHMENT	TS VISITS SPECIALIST	REPORT		
	+ NEW LATE COMPLICAT			

osinters Consulting 360°

Outpatient Appointment Scheduling

 \times

REMOVAL

+ NEW VISIT

ـ

ATTACHMENTS

₽.

 \checkmark

VISITS

Q Search...

Ρ

SPECIALIST I

	Start *	 S • 11/19/2022, 4:00 PM ▶ × ▼ 	
	End *	 € 11/19/2022, 4:30 PM ▶ ⊗ ▼ 	
	Operator	Vittorio ×	
	Description	Suspect infection	©
	Activity status	Planned 🛛 💌 🕶	
	Type of activity *	Control visit	
	Area of competence	Vascular accesses	
	Unit *	Select	* Q
	Patient provenance *	Structure O Domicile O Other structure	
	Other structure of origin	Select	•
			SAVE CANCEL
REPORT			
L			

SCHEDULED MEDICAL EXAMINATION

Explantation of totally implantable venous access





Device removals and explantation recording

۵.

DRESSINGS

A

COMPLICATION

**

CONSULT

	REMOVAL							
	Removal date	© ·	11/19/2022, 5:0	• × •	Removal operator *	Vittorio		× •
	Reason for removal *	Fine Terapia			-			× -
	Note on removal							
)							
)							
							SAVE	CANCEL
S	X REMOVAL							





Specialist report



One or more textual reports entered by specialists

NEW SPECIALIST REPORT

Data Extraction

Extraction list expandable according to needs grouped by topic

							OSINTERS Consulting 36			
	Types of					MAIN MENU				
	f extractions availab					GENERAL Operators Patients Consultation reque Data extraction	est list	Calendar Sending notifica Receive notifica	ations tions	
= 💟 2care	Type some to search for .		Q				MENU +	HELLO		
Calendar	DATA EXTRACTION	NS AND S	TATISTICS							
Vascular access list	Q Sear	Data inizio	1/1/2021		Data fine					
Vascular access request list	✓ GENERALE	Reparto			Operatore	, .				
Operators	Dressings	Select	•	Q	Select		•			
Data extraction	Consultations	Tipo catetere Select	.		·					
Dressing material kit	✓ ACCESSI VASCOLARI			Mult	liple filter area		<u>₽</u> - Q	Search		
lew request for vascular access	Vascular access		Descrizione		Valore1		Valore2	- Valore3	=	
iew removal	Vascular access requests	0	0		0		0	0		
ate Complications	Statistical report	1	NUMERO ACCESSI VASCOLARI (TUTTI - ATTIVI - RIMOSSI)		527		346	181	_	
Departments		2	TEMPO MEDIO POSIZIONAMENTO		21.67		1			
		3	MEDIA GIORNI CATETERE (SOLO RIMOSSI)		113.1					
		4	INCIDENZA CR-BSI/1000 GIORNI CATETERE		0.00590				_	
		5	CONTEGGIO IMPIANTI PER CHEMIOTERAPIA	Resu	lts area exporta	ble to Excel				
		б	DELTA MISURA STIMATA/INTRODOTTA		1.45					
		7	ETA MEDIA PAZIENTI		72.19					
		8	NUMERO CONSULENZE		278					
		9	NUMERO MEDICAZIONI		2245					
		10								
		11	INDAGINESTRUMENTALI		Foodoppler		201		-	